## COMO SUMINISTRAR

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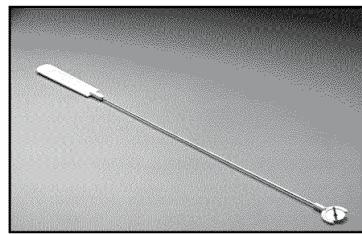
Manufactured By:  
ARMM, Inc.  
17744 Sampson Lane  
Huntington Beach, CA 92647 USA  
Telephone: 714.848.8190  
Fax: 714.848.6141

Hemorrhage Occluder™ Pin  
with Applicator

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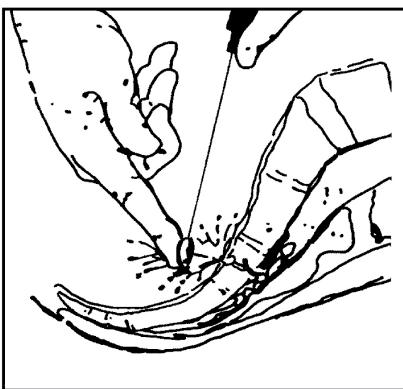
## ENGLISH

### DESCRIPTION

The Hemorrhage Occluder Pin is designed to control severe presacral hemorrhage during pelvic surgery.

### INDICATIONS

The Hemorrhage Occluder Pin is indicated for the control of localized severe hemorrhage from the presacral area during surgery. The pin should be used when other techniques (cautery, suture, clamping, etc.) are judged ineffective.

### CONTRAINDICATIONS

- Bleeding deemed controllable by direct suture or electrocautery.
- Severe instability of the patient as measured by severe hemorrhage or hypotension.
- Bleeding further than 2 cm from the midline. Bleeding that appears to originate from a sacral neural foramen or a vital structure such as a ureter, rectum or vagina.
- Diffuse hemorrhage related to a systemic coagulation disorder not controllable by fingertip pressure against the anterior surface of the sacrum.

### INSTRUCTIONS FOR USE\* (See Figure 1)

- Apply direct fingertip pressure to the bleeding site to stop the bleeding.
- When the bleeding is controlled, ready the preloaded sterile applicator for use by bending the malleable shaft as desired.
- Clear the surgical field of blood and clot and retract adjacent organs for optimal visibility of the presacral area.
- Rapidly withdraw the hemostatic finger and place the occluder pin directly over the bleeding area.
- Apply fingertip pressure to the head of the pin, pushing it into the bone. Apply direct and steady pressure until the pin is fully seated with the head of the pin flush with the bony cortex.
- Consider placement of a second pin if the bleeding continues and appears to be emanating from a second site. Avoid overlapping pins heads.

### HOW SUPPLIED

- Each pin is pre-loaded in its own applicator, individually packaged and STERILE.
- Two pins are supplied per box.
- Single use only.

### COMPLICATIONS

Complications may result from the use of this product. They depend upon the patient's degree of intolerance to any foreign object implanted in the body.

### PRODUCT INFORMATION DISCLOSURE

Surgin Inc. has exercised reasonable care in the choice of materials and manufacture of this product. Surgin Inc. excludes all warranties of MERCHANTABILITY OR FITNESS. Surgin Inc. shall not be liable for any INCIDENTAL OR CONSEQUENTIAL LOSS, DAMAGE OR EXPENSE ARISING directly or indirectly from the use of this product. Surgin Inc. neither assumes nor authorizes any other person to assume for it, any other or ADDITIONAL LIABILITY OR RESPONSIBILITY in connection with the device.

**CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.**

Reuse of this device may result in operational failure or injury due to a dull pin tip resulting in an insecure anchor, and / or risk of infection to the patient from Re-sterilization.

\*A variety of surgical techniques may be used when implanting the Hemorrhage Occluder Pin. Therefore, the surgeon is advised to use the method which his own practice and discretion dictate to be best for the patient.

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